

Roger Modes – Quick Guide



Roger Touchscreen: User Modes

This guide has been designed to support the use of your Roger Touchscreen. It will provide you a quick overview on how to use the mode setting.

Understand the TSM Modes:

Automatic microphone modes

For optimal use with minimal manual interaction, Roger TSM features an automatic microphone function which conveniently switches from an individual talker to a small group interaction mode, based on the orientation of the device. This allows students and teachers to keep up with the ever-changing learning environments.


Lanyard Mode: (Direct instruction)

The TSM is worn around the neck, the distance from the mouth to the top of the TSM should be around 20cm.



Small Group Mode: (Small group work)


Placing the TSM on flat in the middle of the table, this will automatically activate its 360-degree (omnidirectional) microphone setting, which picks up voices from all around. When the small group activity has ended, place TSM back around neck for one speaker usage.

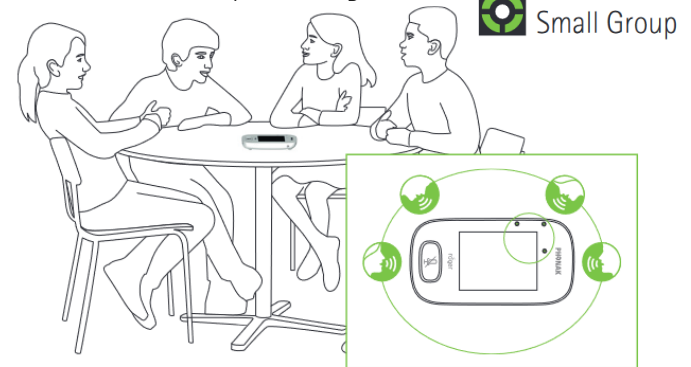

Pointing Mode: (Handheld use)

Teacher or student can point at the person speaking, within 4 meters away. Good for class presentations when multiple people are speaking.

Note: Pointing mode is deactivated by default. To activate it, tap **Settings/Pointing/Activate**

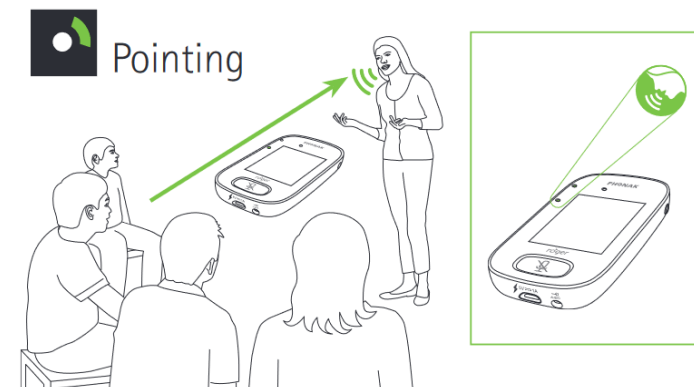

TSM Status Bar: (Understanding the symbols)

It's important to understand the symbols that are shown at the top of the TSM.

	Symbols	Meaning
Always visible		Battery status/Charging/ Low battery
		Remaining use time
		Microphone mode: Lanyard/ Small Group/Pointing
		Network mode: Network/ SubNet
Visible only in use		Audio: Input/Output/Boom
		USB cable connected
		Network role: Stand-alone/ Primary/Secondary